



With Bova Compounding's Click-ur-Dose applicator, medicating your pet has never been easier.

What follows is a brief instruction sheet on how to properly use the Click-ur-Dose applicator to ensure accurate dosing of your animal's medication.

Each applicator contains around 1 month of medication. It is important to note that you should carefully read all material provided and to follow the instructions on the prescription label closely.

**Step 1.** Before the first use be sure to prime the applicator. Rotate the end of the applicator several times until the gel begins to come out of the tip. This will ensure that the pen is ready for accurate dosing.

**Step 2.** Before wiping off the medication, make sure the 2 indicator arrows are pointing towards each other, wait seven to ten seconds and then wipe off the excess medication. Your transdermal gel applicator is ready to use. Don't worry about losing any transdermal gel; we overfill each applicator to compensate for any lost medication.

**Step 3.** Rotate the end of the applicator one full rotation (8 clicks) until the indicator arrows are facing each other again. Once again wait seven to ten seconds so that the correct amount of gel is discharged from the tip. We recommend clicking the applicator in a separate room from your pet, so that he or she does not negatively associate the noise of the applicator with receiving the medication.

**Step 4.** To apply the gel, either use the soft silicone tip of the applicator, or remove the gel with a gloved finger and rub it into your pet's ear. Rub in thoroughly as you would do with lotion.

**Step 5.** Between uses, store the applicator at room temperature and away from light and make sure the lid is secured properly. Do not refrigerate the applicator; doing so may damage the medication.

**Step 6.** Prior to applying later doses, take a damp cloth and wipe away any residue or build-up on the animal's ear.

**Step 7.** Re-order when the plunger of the applicator is near the end of the barrel.

Remember, not all medication works well transdermally. It is important to discuss the positives and negatives of transdermal delivery with your veterinarian or pharmacist when considering this option.

If you have any further questions please call us on 02 9525 3044 to speak with one of our pharmacists, they'll be more than happy to discuss this option with you.